





Open software to patient's volume and highlight the image you wish to export

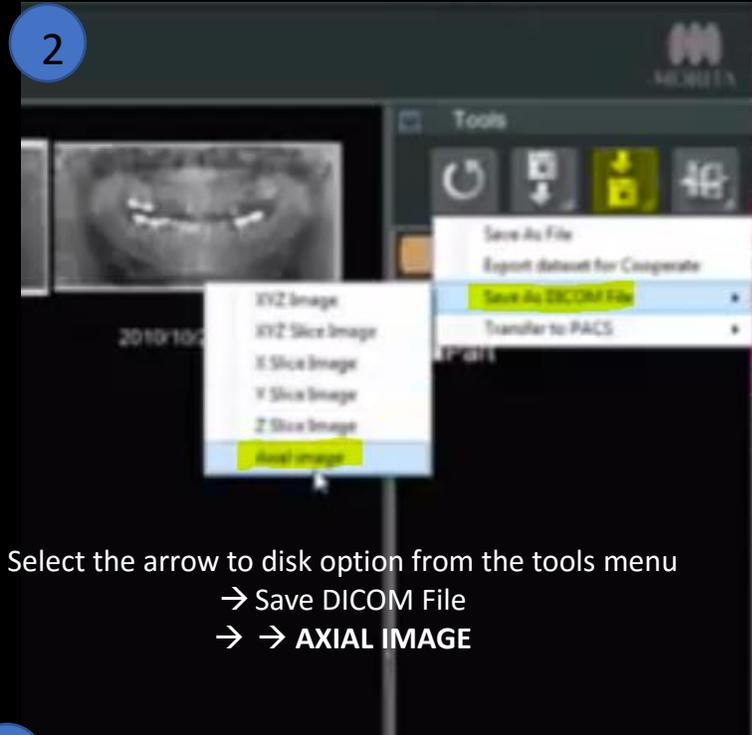


Select the arrow to disk option from the tools menu
→ Save DICOM File
→ → AXIAL IMAGE

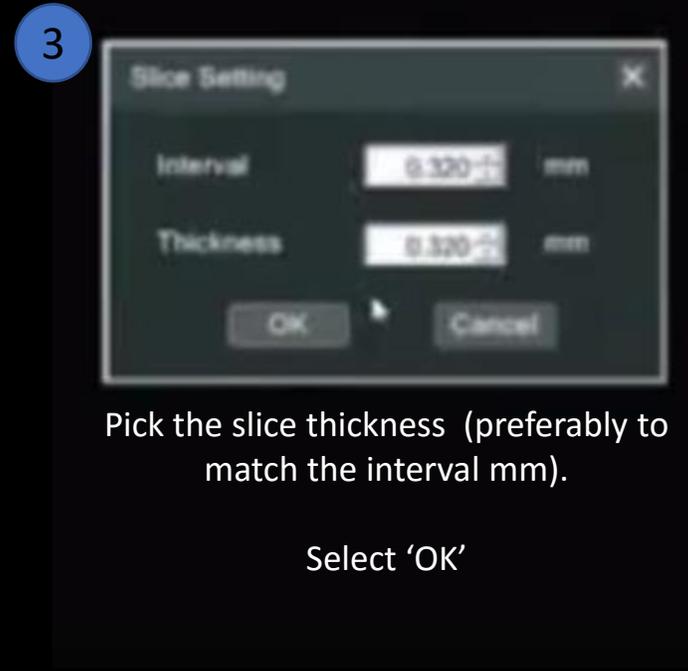


Pick the slice thickness (preferably to match the interval mm).

Select 'OK'

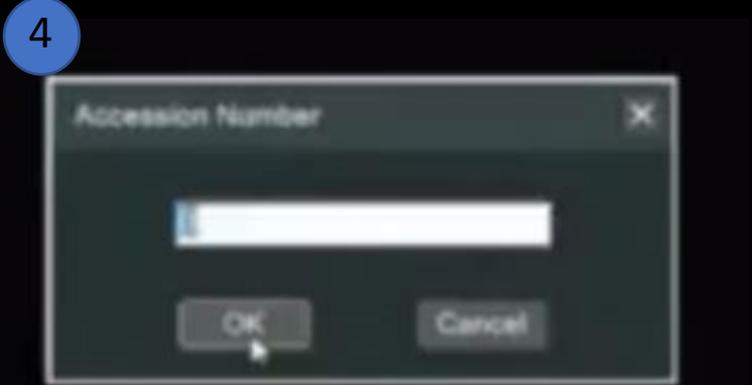

Select 'OK'.

The file will export to the predetermined folder you have established for your exports from this software (often a shortcut on the desktop).

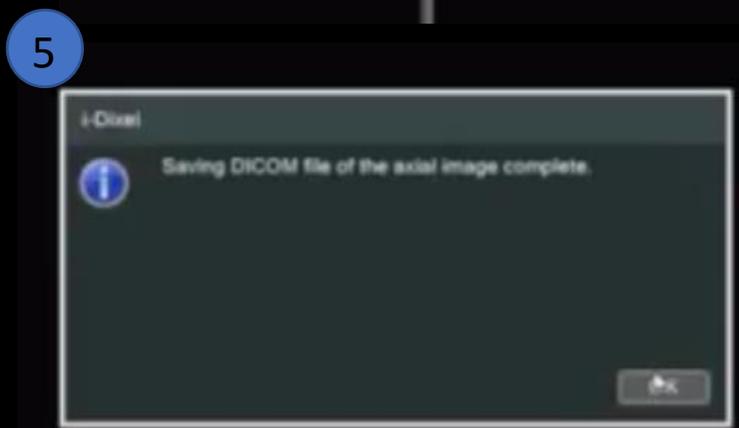

Saving DICOM export in axial image complete
Select 'OK'

- 6
1. Locate the folder on your desktop where the export went.
 2. Make sure the files listed are in "SLZ+####" format
 3. On the export folder containing the files, Right click → Send to → compressed zip folder.
 4. The zipped file is the file that needs to be submitted to the "Patient Files" section of the TELEDENT website.

Please do not submit the OneVolume Viewer