


## Exporting from EZ3D-I (VaTech)


1. On your desktop, Right click once → “New” → “Folder” and Name the folder with the patient’s Name
2. Launch software and patient scan
3. Go to “Main Menu” then “Export”



- 4.
5. Ensure the following radial buttons/boxes are checked:
  - a. LOCAL DISK, DCM Files, Compression is set to “Uncompressed” or “Lossless” if uncompressed is not an option. DO NOT CHECK THE “VIEWER” BOX!!!

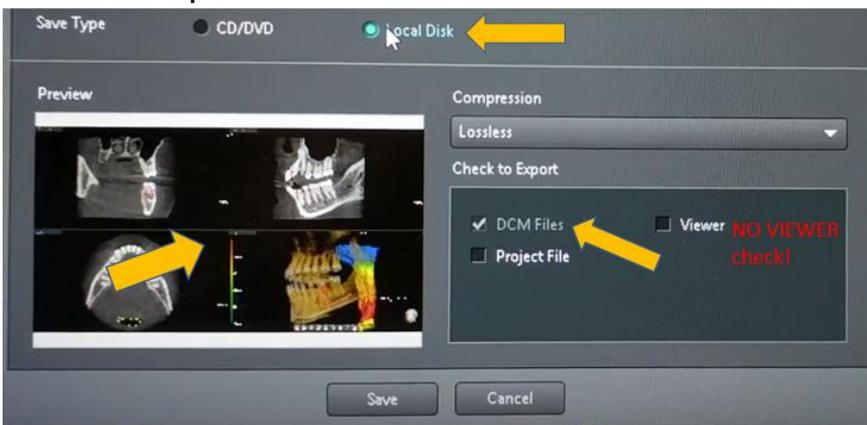

- 6.
7. Select “Save”
8. Browse to your Desktop and Find the folder with your patient’s name that you created in step #1. Select the Folder

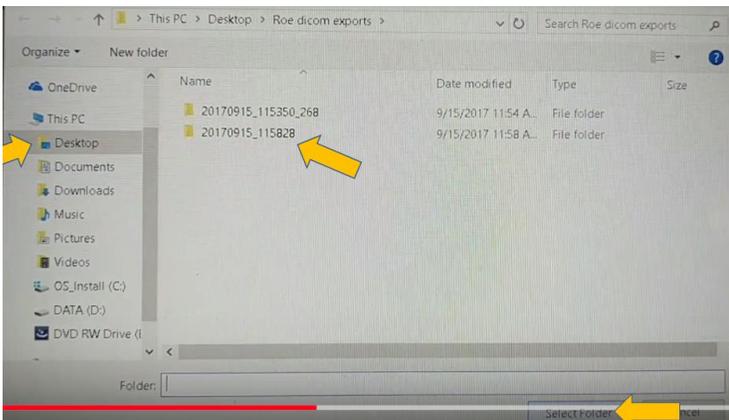

- 9.
10. The scan volume will export and you will be prompted with a notification box upon completion of the export.
11. Return to your desktop and find the patient’s folder from step 1
12. Right Click on the folder → Send to → Compressed Zip Folder
13. The zipped folder (with a zipper on it) is the folder that needs to be submitted to the Teledent site for interrogation and reporting.